

ALADDIN

Optical Biometry and
Topography System



Aladdin addresses the limitations of conventional biometers



*) Available in Aladdin HW3.0 only

The complete picture - enhanced.

With the combination of an optical biometer and full corneal topographer, Topcon pioneered the concept of “the complete picture” in IOL power calculation. Now the complete picture has been enhanced with the addition of the Barrett Universal formula as a standard component of the Aladdin.*

The final optical result and patient’s satisfaction are paramount in today’s cataract surgery. With the incorporation of the latest in IOL calculation formulae, the Aladdin remains at the forefront of IOL calculation technology.

1 Instrument; 9 Functions

- » Axial Length
- » Keratometry
- » Anterior Chamber Depth
- » Central Corneal Thickness*
- » Lens Thickness*
- » Corneal Topography
- » Corneal Wave-front Analysis
- » Pupillometry
- » White-to-white

Aladdin features

Fully integrated patient database

- » Patient search function
- » Input post refraction data

Easy acquisition 9-in-1

- » Pre-op input of lens and vitreous body

Conventional IOL calculation formulae

- » SRK II, SRK/T , Hoffer Q, Holladay 1, Haigis
- » Multiple surgeon pre-settings
- » Export to Olsen PhacoOptics® PC software
- » ULIB database compatible
- » Database customizable

Post refractive IOL calculation formulae

- » Camellin-Calossi, Shammas (no history)

Generic toric IOL calculation

- » Toric IOL rotation simulator
- » Abulafia-Koch Astigmatism Formula

Barrett IOL Calculator Suite*

- » With the Barrett Rx, the Barrett Toric Calculator Formula, the Barrett True K and the Barrett Universal II formulae.

Topography

- » Full featured corneal mapping
- » Accurate corneal radii
- » Keratoconus probability index

Corneal wave front (Zernike) analysis

- » Maps (pupil size 2,5 – 7.0 mm)
- » Simulation graphs

Interferometer graphs

- » Axial length
- » Central corneal Thickness*
- » Anterior chamber depth
- » Lens Thickness*

Pupillometry

- » Dynamic, Photopic, Mesopic
- » Decentralization and Latency graph

White-to-white measurement

Reports

- » Biometry report (AL, K, ACD, LT*, CCT* WTW)
- » To USB, shared folder and printer
- » Topography report
- » IOL report
- » Pupillometry

DICOM™ Compliance

The DICOM panel in the Aladdin Connectivity section allows the user to set the needed parameters for the connections to the available DICOM features:

- » Modality Worklist
- » Patient Root Query
- » Storage
- » Storage Commitment



Speed

Point and shoot acquisition, all necessary measurements are taken in under five seconds. Single measurement are supported for even faster ACD, AL or topography, as well as a separate full pupillometry.


Accuracy

Proven interferometry accuracy combined with new technology for keratometry, provide extremely accurate Axial Length and corneal radii information for precise calculation of IOL spherical and toric powers.

Posterior & Anterior interferometry

You get the complete picture for all cataract surgeries. Whether you are performing standard cataract surgery or premium IOL implantation, you will be screening for corneal aberrations, Keratoconus and previous corneal refractive surgery procedures all at once. The Aladdin only requires just one acquisition.




Biometry results are complemented with anterior topography, Zernike analysis and pupillometry. Aladdin also provides anterior measurements such as the Central Corneal Thickness (CCT)*, Anterior Chamber Depth (ACD) and Lens Thickness (LT)*

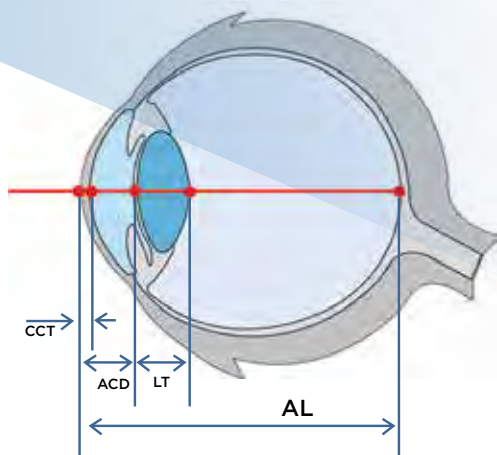

Ease of use

The operator is only three clicks away from printing the Aladdin report. The 10.1 inch full colour touchscreen monitor is very responsive and comfortable to use. The user-friendly interface guides you through the main functions with ease.

Making an acquisition has never been easier. To ensure complete accurate biometry, the Aladdin guides you in focus and alignment with visual color coded signs while taking the acquisition.

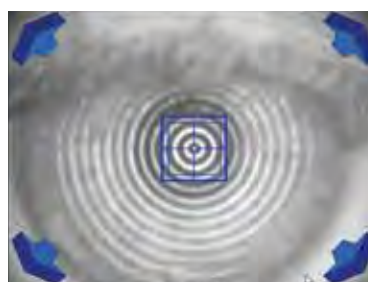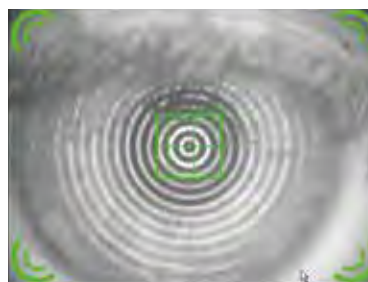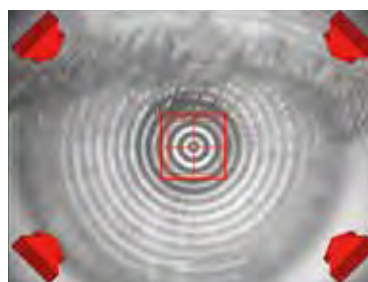

*) Available in Aladdin HW3.0 only

Onboard Barrett Formulae*

Dr. Graham D. Barrett developed the Barrett Formula Suite in 2013. The formulae take into account the posterior cornea, calculate the lens position for each individual patient including consideration of the measured lens thickness rather than an estimation based on patient's age. The Universal II formula developed by Dr Barrett, is a method of predicting IOL power based on Gaussian optics and utilizes this information to calculate the effect of the cylinder power at the cornea.

Dr. Barrett's formula considers the thickness and shape of the IOL implant as well, which provides a more sophisticated way of predicting and translating the cylinder power. The formula is able to predict the posterior corneal curvature for an individual patient utilizing a theoretical model rather than direct measurement. The new version of the Aladdin, accurately measures the lens thickness, an important component of the Barrett formula.



The Aladdin's Barrett Formula Suite includes the Barrett Rx, the Barrett Toric Calculator Formula, the Barrett True K and the Barrett Universal II formulae.

Abulafia-Koch astigmatism cylinder correction for Toric IOL calculations incorporated

The Abulafia-Koch correction formula calculates the estimated total corneal astigmatism based on standard keratometry measurements.



Dr. Adi Abulafia



Dr. Douglas Koch

* Available in Aladdin HW3.0 only



Precise Toric IOL calculation

A robust generic Toric IOL calculator is incorporated into the Aladdin software. This integrated toric IOL calculator saves time and avoids unnecessary mistakes when manually entering data online. IOL toric rotation simulation software calculates the induced spherical and cylindrical power for every five degrees that the toric IOL rotates. Surgically Induced Astigmatism (SIA) and Incision Location (IL) can be inserted by the surgeon and are taken into account for the Toric IOL calculation.

Axial length


Using a low-coherence interferometry system with a super luminescent diode of 830 nm and signal processing, the Aladdin achieves axial length measurement with high signal-to-noise ratio and is able to penetrate even high grade dense cataracts. Axial length measurements can be done on normal eyes as well as on aphakic, pseudo-aphakic and silicone oil-filled eyes.



Anterior biometry

Anterior biometry with the Aladdin makes it possible to measure Central Corneal Thickness*, Anterior Chamber Depth and the Crystalline Lens Thickness*. ACD is measured through interferometry* providing high precision and reproducibility. All interferometry measurements are shown in a single graph quick reference.



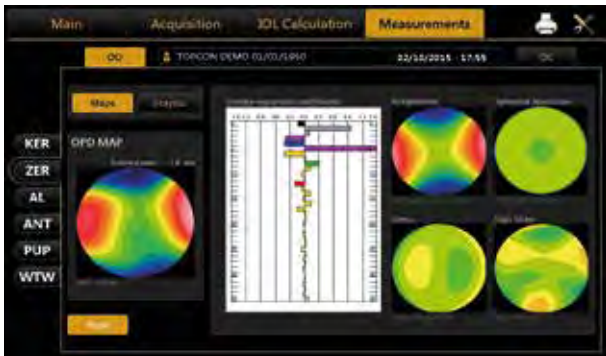


Aberrometry analysis (Zernike)

Zernike analysis of the topographic data provides the Optical Path Difference (OPD) and information on astigmatism, spherical aberrations, higher order aberrations and Coma for pupil sizes of 2.5mm to 7.0mm. When using the actual Spherical Aberration provided by Zernike analysis, you can select the appropriate aspherical IOL with standardised spherical aberration correction according to the patient's individual required spherical aberration.

Keratometry / Topography

Full corneal topography provides substantially more information than conventional central keratometry. Corneal topography data is especially useful in the selection of toric IOLs to quickly differentiate regular and irregular astigmatism as well as corneal aberrations. The Aladdin provides accurate corneal topography obtained from the reflection of a set of 24 Placido rings in combination with a low coherence interferometer.

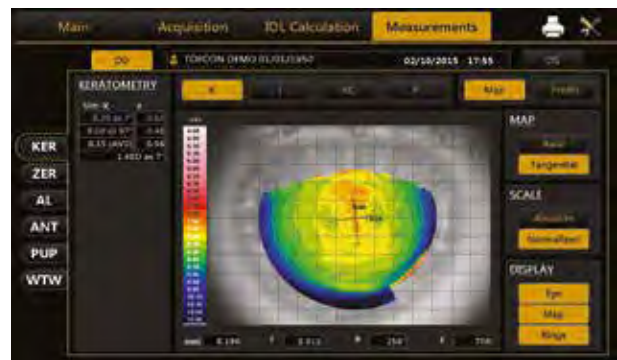

Keratoconus screening

The Aladdin is capable to screen the corneal surface for keratoconus probability. This information provides the surgeon in detail the corneal keratometric indices to assist in making the correct toric IOL selection. The Keratoconus Probability Index is shown in percentage as well as in colour codes.


- Green Not compatible with Keratoconus
- Yellow Suspected Keratoconus
- Red Compatible with Keratoconus

Pupillometry

During Placido evaluation, pupillary response is observed to assess a pseudo Photopic and pseudo Mesopic pupil size, indicating response and normal range of the pupil. Full pupillometry screening assists to evaluate eyes for multifocal IOL implantation or refractive surgery. For any refractive procedure it is important to carefully

evaluate the pupil size in different light conditions to address cases of extremely small or decentered pupils. The Aladdin can perform pupillometry in three different modes:

- » Dynamic
- » Photopic
- » Mesopic

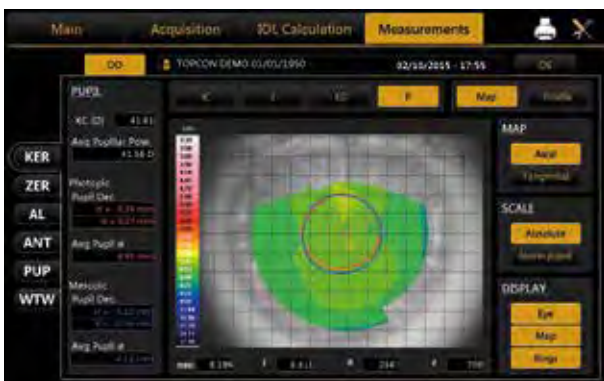


White-to-white

The Aladdin automatically measures white-to-white length which can be manually edited. Reliable white-to-white measurement is used for anterior chamber intraocular lens and sulcus fixated posterior chamber intraocular lens calculation in highly myopic eyes.



Post refractive IOL

In eyes that have previously undergone refractive surgery such as RK, PRK, Lasik, Lasek, LK and PTK, spherical aberrations are often outside the standard values. In these cases the Aladdin provide the Camellin Calossi and the Barrett

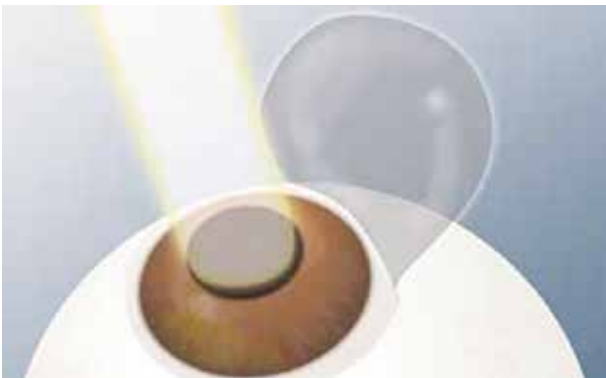

True K formulas. If there is no patient history the Shamma's No-history or the Barrett True K formulas can be used for the correct IOL calculation.



Customizable IOL database

The Aladdin provides a full ULIB database which can be easily upgraded and customized. The surgeon can manually upgrade the A- constant for each individual IOL to obtain even a higher accuracy every time a cataract surgery is performed. Favorite IOL's can be selected and programmed for each surgeon, making IOL selection simple and personalized.



IMAGEnet®6 Compliance

IMAGEnet®6 viewer software

IMAGEnet®6 is Topcon's web based digital software platform for ophthalmic imaging, capable of acquiring, displaying, enhancing, analyzing, and saving digital images and reports obtained with a variety of Topcon devices such as the Aladdin. IMAGEnet®6 provides flexibility of viewing Biometry, Keratometry, Pupillometry data and all available Aladdin exported reports in a network environment. There are a variety of software configurations available. Additional components can be added according to your clinic's needs.



Screenshot of Biometry data in IMAGEnet®6

Topcon's Cataract Workstation



KR-800S Auto kerato-refractometer with subjective function

- 1 VA check far vision
- 2 VA check near vision
- 3 VA check glare condition
- 4 VA check contrast condition
- 5 Grid test (AMD screening)
- 6 VA simulation Premium IOL



KR-800S



Pre-Operative
Subjective Refraction
and Pre-OP-diagnostics

Cataract surgery quality control

Topcon's Cataract Workstation

Visual acuity (VA) is the most common clinical measure of the quality of cataract surgery. It is how the success of surgery is measured and it is therefore, critical that it is measured correctly. Measurement of VA must be standardized and systematic. Topcon's KR-800S Auto Kerato-Refractometer with subjective VA check will do exactly that. With the KR-800S the VA can be subjectively tested before and after cataract surgery. With the unique features of the KR-800S such as "Glare" test and "Contrast" test, the progression and differentiation of nuclear cataract from cortical cataract can be evaluated.


Premium IOL simulation

The KR-800S offers a Spherical Equivalent mode, which can simulate the benefit of a premium (toric) IOL, to encourage the patient to reach an even higher post-operative VA. The subjective VA test for near vision will demonstrate to the patient the benefits of a Multifocal IOL.

ALADDIN



Pupillography
Topography
Biometry inkl. K1 & K2
IOL Calculation

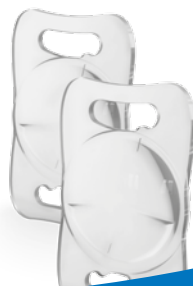

Cataract
Surgery

KR-800S



Post-Operative
Subjective Refraction
and Post-OP-diagnostics

Reports measurement summary



Topcon Europe Medical bv

Patient : TOPCON DEMO

Surgeon : Surgeon Generic

Patient ID :

Exam Date : 02/10/2015 - 17:55

Date Of Birth : 01/01/1950
(mm/dd/yyyy)

(mm/dd/yyyy)

OD



OS

Phakic

Normalized Axial Map

Normalized Axial Map

Phakic

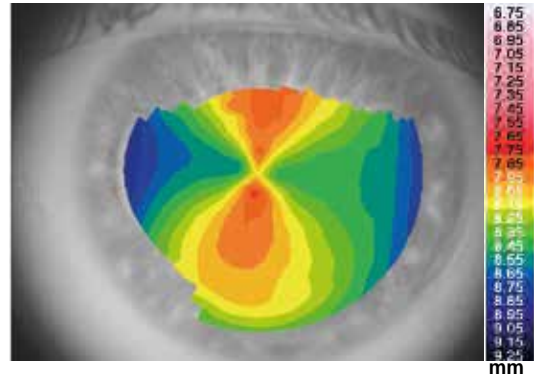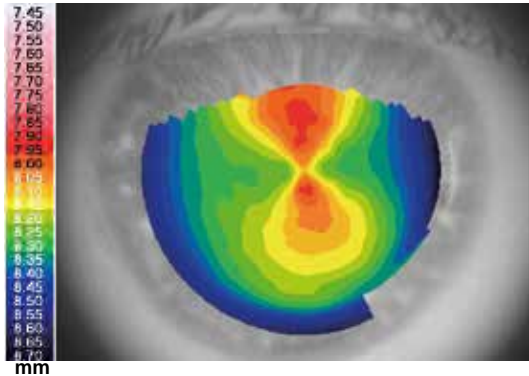

Measurement Summary

AL	23.73 mm	K1	8.28 mm@	8°	AL	23.93 mm	K1	8.51 mm@	173°
ACD	3.14 mm	K2	8.00 mm@	98°	ACD	3.21 mm	K2	7.90 mm@	83°
LT	4.04 mm	CCT	0.544 mm		LT	4.00 mm	CCT	0.556 mm	
WtoW	11.70 mm	Dec	(-0.22, -0.29)		WtoW	11.92 mm	Dec	(0.40, -0.07)	

Keratorefractive Indices

CYL 3 mm	-1.44 D	Ax:	7°	CYL 3 mm	-3.18 D	Ax:	172°
CYL 5 mm	-1.46 D	Ax:	8°	CYL 5 mm	-3.16 D	Ax:	172°
SD	SAI	e	Kc	SD	SAI	e	Kc
0.36 D	0.47 D	0.49	41.61	0.44 D	0.55 D	0.39	41.40

Keratoconus Screening

AK	AGC	SI	p	AK	AGC	SI	p
43.03 D	0.90 D/mm	-0.50 D	0%	43.46 D	0.68 D/mm	-0.40 D	0%

Pupil Data

Photo: Diam	3.95 mm	Dec	0.35 mm; 168°	Photo: Diam	4.24 mm	Dec	0.21 mm; 343°
Meso: Diam	4.11 mm	Dec	0.32 mm; 187°	Meso: Diam	4.45 mm	Dec	

Zernike Analysis 5 mm

OPD	Coma	Sph. Ab.	OPD	Coma	Sph. Ab.
rms: 0.80 µm	rms: 0.15 µm	rms: 0.10 µm	rms: 1.43 µm	rms: 0.07 µm	rms: 0.14 µm



Reports IOL calculation



Patient : TOPCON DEMO

Patient ID :

Date Of Birth : 01/01/1950
(mm/dd/yyyy)

Topcon Europe Medical bv

Surgeon : SURGEON GENERIC

Exam Date : 02/10/2015 - 17:55
(mm/dd/yyyy)

OD

Phakic

Data Measurements n: 1.3375

Aladdin Optical

AL : 23.73 mm K1 : 8.28 mm @ 8°
ACD : 3.14 mm K2 : 8.00 mm @ 98°
LT 4.04 mm CYL : -1.45 D ax 8°
CCT 0.544 mm

Target Refraction: 0

Oculentis
L-313

SRK/T	
IOL(D)	REF(D)
20.50	0.83
21.00	0.47
21.50	0.10
22.00	-0.27
22.50	-0.64

IOL @ Target A = 118.100
21.64

Oculentis
LS-313 MF30

SRK II	
IOL(D)	REF(D)
21.00	0.77
21.50	0.37
22.00	-0.03
22.50	-0.43
23.00	-0.83

IOL @ Target A = 118.600
21.97

Oculentis
LU-313 MF30T

Haigis	
IOL(D)	REF(D)
21.50	0.58
22.00	0.21
22.50	-0.16
23.00	-0.54
23.50	-0.92

IOL @ Target A0 = 0.870
22.28 A1 = 0.400
A2 = 0.100

Oculentis
LS-412Y

Hoffer Q	
IOL(D)	REF(D)
21.00	0.86
21.50	0.51
22.00	0.16
22.50	-0.20
23.00	-0.56

IOL @ Target pACD = 5.070
22.22

Oculentis
LU-800 RZI

Holladay I	
IOL(D)	REF(D)
19.00	0.90
19.50	0.52
20.00	0.13
20.50	-0.25
21.00	-0.65

IOL @ Target SF = 0.310
20.17

OS

Phakic

Data Measurements n: 1.3375

Aladdin Optical

AL : 23.93 mm K1 : 8.51 mm @ 173°
ACD : 3.21 mm K2 : 7.90 mm @ 83°
LT 4.00 mm CYL : -3.06 D ax 173°
CCT 0.556 mm

Target Refraction: 0

Oculentis
L-313

SRK/T	
IOL(D)	REF(D)
20.50	0.67
21.00	0.31
21.50	-0.06
22.00	-0.43
22.50	-0.81

IOL @ Target A = 118.100
21.42

Oculentis
LS-313 MF30

SRK II	
IOL(D)	REF(D)
21.00	0.62
21.50	0.22
22.00	-0.18
22.50	-0.58
23.00	-0.98

IOL @ Target A = 118.600
21.77

Oculentis
LU-313 MF30T

Haigis	
IOL(D)	REF(D)
21.00	0.81
21.50	0.45
22.00	0.08
22.50	-0.30
23.00	-0.67

IOL @ Target A0 = 0.870
22.10 A1 = 0.400
A2 = 0.100

Oculentis
LS-412Y

Hoffer Q	
IOL(D)	REF(D)
21.00	0.72
21.50	0.37
22.00	0.01
22.50	-0.35
23.00	-0.71

IOL @ Target pACD = 5.070
22.02

Oculentis
LU-800 RZI

Holladay I	
IOL(D)	REF(D)
19.00	0.76
19.50	0.38
20.00	-0.01
20.50	-0.40
21.00	-0.80

IOL @ Target SF = 0.310
19.99

Reports generic toric IOL calculation



Topcon Europe Medical BV

Patient : TOPCON DEMO

Surgeon : Surgeon Generic

Patient ID : ~

Exam Date (dd/mm/yyyy) : 10/02/2015 - 17:55

Date Of Birth (dd/mm/yyyy) : 01/01/1950

OS
Phakic

Measures (Aladdin Optical)

K1:	39.64 D	AL:	23.93 mm	LT:	4.00 mm	WTW:	11.98 mm
K2:	42.71 D	ACD:	3.21 mm	CCT:	0.556 mm	WTW Dec	(0.40,-0.07) mm
CYL:	-3.06 D @ 173°						
n:	1.3375						

Toric IOL


Target Refraction: **0.00 D** SIA: **0.00 D** IL: **83°**

Toric IOL: Oculentis LS-313 T3
21.50 D (S.E.) 3.00 D @ 81°

LF = 1.412, A constant = 118.100

Cylinder Power: IOL Plane 3.00 D ~ Corneal Plane 2.20 D

Predicted refraction:
0.07 D sph. -0.03 D @ 81°

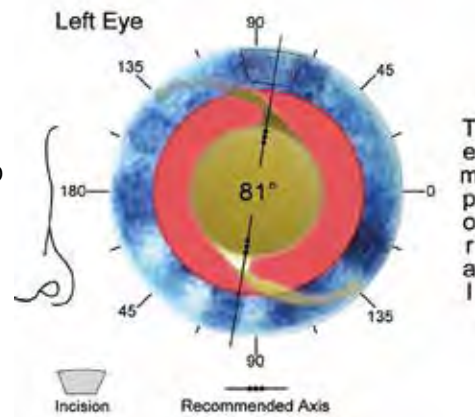

IOL Power (S.E.)	Refraction (S.E.)	IOL submodel	IOL toricity	Residual astigmatism
20.50 D	0.81 D	LS-313 T1 LS-313 T2 LS-313 T3 LS-313 T4 LS-313 T5	1.50 D	-1.07 D @ 171°
21.00 D	0.43 D		2.25 D	-0.52 D @ 171°
21.50 D	0.06 D		3.00 D	-0.03 D @ 81°
22.00 D	-0.33 D		3.75 D	-0.58 D @ 81°
22.50 D	-0.71 D		4.50 D	-1.13 D @ 81°



Reports generic toric IOL calculation



TORIC IOL



Patient Information

Patient TOPCON DEMO	Surgeon SURGEON GENERIC	OS
Patient ID	Clinic Topcon Europe Medical bv	
Date of Birth 01/01/1950 <small>mm/dd/yyyy</small>	Exam Date 02/10/2015 - 17:55 <small>mm/dd/yyyy</small>	

Biometry Data

AL (mm)	23.93	LT (mm)	4.00	K1 (mm)	8.51	CYL (D)	-3.06@173°
ACD (mm)	3.21	CCT (mm)	0.556	K2 (mm)	7.90	n	1.3375

Surgical Pre Op Data

SEQ (D)	23.00	SIA (D)	0
Formula	Holladay I	IL (°)	83

SF = 1.980

Expected Post Op Cornea

K1 Post (mm)	8.51	K2 Post (mm)	7.90
CYL Post (D)	-3.06 @ 173°		

Toric IOL

Lens Model Alcon AcrySof SN6AT6	
Spherical Power 21.50 D	Cylindrical Power 3.75 D
Sph. Equiv. Power 23.38 D	Axis Of Placement 83°
Expected Refraction -0.02D -0.44 D @ 173°	
Lens	Residual Astigmatism
AcrySof SN6AT4 (22.00D 2.25C)	-1.48 D @ 173°
AcrySof SN6AT5 (21.50D 3.00C)	-0.96 D @ 173°
AcrySof SN6AT6 (21.50D 3.75C)	-0.44 D @ 173°
AcrySof SN6AT7 (21.00D 4.50C)	-0.08 D @ 83°
AcrySof SN6AT8 (20.50D 5.25C)	-0.60 D @ 83°

Toric IOL Placement

OS

Nasal **Temporal**

Quantity	1
----------	----------

Notes

--

1.4.0

Reports generic toric IOL calculation



Topcon Europe Medical bv

Patient : TOPCON DEMO

Surgeon : Surgeon Generic

Patient ID :

Exam Date : 02/10/2015 - 17:55
(mm/dd/yyyy)

Date Of Birth : 01/01/1950
(mm/dd/yyyy)

OD

OS

Phakic

Phakic

Axial length values							
Comp. AL: 23.73 mm				Comp. AL: 23.93 mm			
AL		AL		AL		AL	
23.79 mm				23.95 mm			
23.77 mm				23.91 mm			
23.72 mm				23.85 mm			
23.73 mm				23.93 mm			
23.73 mm				23.96 mm			
23.72 mm				23.94 mm			
Value Corneal Curvature							
KER: 8.28/8.00 mm CYL: -1.45 D Ax 8°				KER: 8.51/7.90 mm CYL: -3.06 D Ax 173°			
K1: 8.28 mm @ 8°		40.74 D		K1: 8.51 mm @ 173°		39.64 D	
K2: 8.00 mm @ 98°		42.19 D		K2: 7.90 mm @ 83°		42.71 D	
CYL: -1.45 D ax 8°				CYL: -3.06 D ax 173°			
ACD value							
ACD: 3.14 mm				ACD: 3.21 mm			
3.14 mm				3.21 mm			
LT value							
LT: 4.04 mm				LT: 4.00 mm			
4.04 mm				4.00 mm			
CCT value							
CCT: 0.544 mm				CCT: 0.556 mm			
White to White							
WTW 11.70 mm Dec (-0.22 mm, -0.29 mm)				WTW 11.92 mm Dec (0.40 mm, -0.07 mm)			

Reports generic toric IOL calculation



Topcon Europe Medical bv

Patient : TOPCON DEMO

Surgeon : Surgeon Generic

Patient ID :

Exam Date : 02/10/2015 - 17:55
(mm/dd/yyyy)

Date Of Birth : 01/01/1950
(mm/dd/yyyy)

Dynamic Pupillography

OD

Diameter (mm)

Min	Max
3.48	4.98



Center (mm)

Mean	Std Dev
x= -0.27	0.07
y= 0.02	

OS

Diameter (mm)



Min	Max
3.27	4.78



Center (mm)

Mean	Std Dev
x= 0.25	0.08
y= -0.04	

Latency

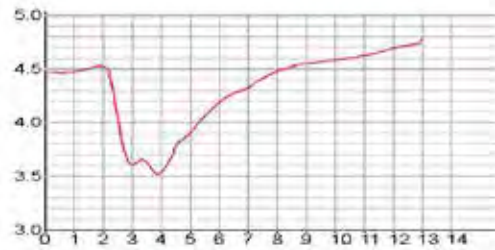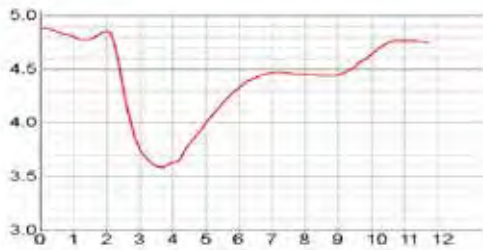

Static Pupillography

Diameter (mm)

	Mesopic	Photopic
Mean	4.57	3.80
Std Dev	0.09	0.09

Center (mm)



	Mesopic	Photopic
X	-0.33	-0.27
Y	0.04	-0.01

Diameter (mm)

	Mesopic	Photopic
Mean	4.60	3.71
Std Dev	0.09	0.10

Center (mm)

	Mesopic	Photopic
X	0.25	0.21
Y	-0.15	-0.09

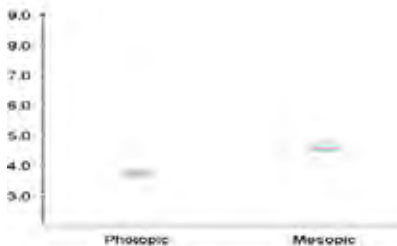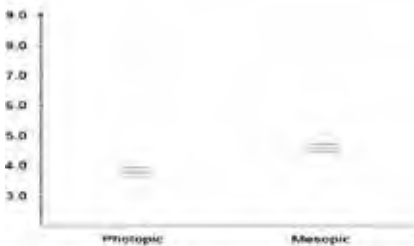

Specifications

ALADDIN HW2.0

ALADDIN HW3.0

Measurement range for IOL		
Axial Length (Interferometry)	Super luminescent diode 830nm, 15,00mm - 38,00mm	
Corneal Radii	3,30mm - 37,50mm / 9,00D - 101,50D	5,00mm - 12,00mm / 28,00D - 67,50D
Anterior Chamber Depth measurement	slit illumination 1,50mm - 5,50mm	interferometer 1,50mm - 6,50mm
White-to White measurement	6,00mm - 18,00mm	8,00mm - 14,00mm
Pupillometry	Dynamic, Photopic & Mesopic, pupil size 0,50mm - 10,00mm	
Lens Thickness (interferometry)	no	1,50mm - 6,50mm (phakic) 0,50mm - 3,50mm (pseudo-phakic)
Central Corneal Thickness measurement (interferometry)	no	0,300 mm - 0,800 mm
On-board calculation formulas		
IOL formulas	Haigis, Hoffer Q, Holladay 1, SRK*II & SRK*IT, Barrett, Universal II	
Post Refractive IOL formulas	Camellin Calossi & Shammas No History Olsen PhacoOptics® connectivity, Barrett True K	
Placido Topography specifications		
Keratoscopic Cone (topographic map)	24 rings on a 43 dpt sphere, working distance 80 mm	
Points analysed	over 100.000	
Points measured	over 6.000	
Cornea coverage	up to Ø 9,8 mm (on a 8 mm sphere) 42,20D with n=1.3375	
Guided focus system	yes	
Keratoconus screening		
Apical Curvature	yes	
Apical Gradient of Curvature	yes	
Symmetry index	yes	
Kpi (Keratoconus probability index)	yes	
Software features		
Toric IOL calculator	Generic Toric IOL, Oculentis Toric IOL	
Zernike analysis	Pupil size 2,5 mm - 7,0 mm	
Print to	USB printer, Network printer, PDF to shared network folder & PDF to USB drive	
Instrument Specifications		
Display	10.1" touch screen	
Storage	At least 320 GB HDD	At least 320 GB HDD + 32 GB SSD
Operating system	Windows 7 Embedded OS	
Processor	AMD G-T56N	
Internal memory	2GB RAM	
Power input	AC 100 - 240V 50 - 60Hz	
Dimensions	320 mm (W) x 490 mm (H) x 470 mm (L)	
Weight	18 kg	
Connections	1 x LAN, 2 x USB	
Supports	USB Barcode scanner, External USB keyboard / mouse	
Marking	CE, ETL	
Reports		
Aladdin report	yes	
Measurement overview	yes	
Pupillometry	yes	
IOL	yes	
Generic Toric IOL	yes	
Oculentis Toric IOL	yes	

IMPORTANT

Subject to change in design and/or specifications without advanced notice.

In order to obtain the best results with this instrument, please be sure to review all user instructions prior to operation.



Topcon Europe Medical B.V.
Essebaan 11; 2908 LJ Capelle a/d IJssel; P.O. Box 145;
2900 AC Capelle a/d IJssel; The Netherlands
Phone: +31-(0)10-4585077; Fax: +31-(0)10-4585045
E-mail: medical@topcon.eu; www.topcon-medical.eu

Topcon Danmark
Praestemarksvej 25; 4000 Roskilde, Danmark
Phone: +45-46-327500; Fax: +45-46-327555
E-mail: info@topcon.dk
www.topcon.dk

Topcon Scandinavia A.B.
Neogatan 2; P.O. Box 25; 43151 Mölndal, Sweden
Phone: +46-(0)31-7109200; Fax: +46-(0)31-7109249
E-mail: medical@topcon.se; www.topcon.se

Topcon España S.A.
HEAD OFFICE; Frederic Mompou, 4;
08960 Sant Just Desvern; Barcelona, Spain
Phone: +34-93-4734057; Fax: +34-93-4733932
E-mail: medica@topcon.es; www.topcon.es

Topcon Italy
Viale dell' Industria 60;
20037 Paderno Dugnano, (MI) Italy
Phone: +39-02-9186671; Fax: +39-02-91081091
E-mail: info@topcon.it; www.topcon.it

Topcon France
BAT A1; 3 route de la révolte, 93206 Saint Denis Cedex
Phone: +33-(0)1-49212323; Fax: +33-(0)1-49212324
E-mail: topcon@topcon.fr; www.topcon-medical.fr

Topcon Deutschland GmbH
Hanns-Martin-Schleyer Strasse 41;
D-47877 Willich, Germany
Phone: (+49) 2154-885-0; Fax: (+49) 2154-885-177
E-mail: info@topcon-medical.de; www.topcon-medical.de

Topcon Polska Sp. z o.o.
ul. Warszawska 23; 42-470 Siewierz; Poland
Phone: +48-(0)32-670-50-45; Fax:
+48-(0)32-671-34-05
www.topcon-polska.pl

Topcon (Great Britain) Ltd.
Topcon House; Kennet Side; Bone Lane; Newbury
Berkshire RG14 5PX; United Kingdom
Phone: +44-(0)1635-551120; Fax: +44-(0)1635-551170
E-mail: medical@topcon.co.uk; www.topcon.co.uk

Topcon Ireland
Unit 276, Blanchardstown; Corporate Park 2
Ballycoolin; Dublin 15, Ireland
Phone: +353-18975900; Fax: +353-18293915
E-mail: medical@topcon.ie; www.topcon.ie